

Competency for TURBO

1. To power On/Off- left side of machine.
2. To enter patient information
 - a. Press **PATIENT** key- on RT. Side
 - b. Press **NEW PATIENT** which is the 1st soft key
 - c. Type first, (ENTER) last name, ID, etc. Then select **DONE** (last soft key) or press **PATIENT** key to get out.
 - d. Choose either the Nerve or Vascular Exam Setting.
3. Frequency can be changed by selecting RES, GEN, PEN (1st Soft key)
 - a. RES = best resolution= highest frequency= most breasts and Vascular
 - b. GEN = middle frequency = some larger breasts
 - c. PEN = best penetration but loses resolution = lowest frequency
4. To adjust GAIN (brightness or darkness of image) use AUTOGAIN or the 3 gray dials
 - a. Near =Top Half of screen
 - b. Far = Bottom half of screen
 - c. Gain = Overall image
5. **Depth**
 - a. Depth Up Arrow= allows you to image objects that are shallow, Just below the skin
 - b. Depth Down Arrow = allows you to image objects that are deeper in the body
6.
 - c. Press **FREEZE**, then **SAVE** to save image to Memory
 - d. **RETRIEVE** (Rt. Side) allows you to recall **SAVED** images or Clips And **PRINT** or **DELETE** Files.
 - e. Press **REVIEW** (soft key) to view saved images.
 - f. Press **PRINT** to print pictures.
6. **TEXT**- to get cursor to end of the TEXT line, select the Arrow Up key.
7. **CALIPER** allows you to measure in 2 directions. Press **FREEZE**. Then Press **CALIPER**. Use touch pad to move green +, press **SELECT** to activate the opposite end. Press **CALIPER** again to get the X caliper up and repeat. You may also use this to measure the distance from SKIN LINE to the Vessel or target.
8. **TO DISINFECT** the Probe or keyboard.... USE wet disinfectant wipes.
9. **TO clean screen, use damp cloth.**